


# Eroxon®

209834

U111  
FPO  
U111



Read leaflet before use. For more information, scan or go to Eroxon.us

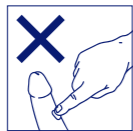

Read the entire Consumer Information Leaflet before using this product. If you have any questions, talk to your doctor before use.

## INDICATION FOR USE

Treatment of erectile dysfunction in males aged 22 years and older.

## WHAT EROXON® IS FOR

Eroxon® is a clinically proven gel treatment for adult men with erectile dysfunction, or ED, which is the inability to get or keep an erection sufficient for satisfactory sexual activity.

Eroxon® helps you get and maintain an erection hard enough for sex.

**Fast Acting** Eroxon® can be applied immediately prior to sexual activity and is clinically proven to help you get an erection within 10 minutes, for physical intimacy.

## CLINICAL PERFORMANCE

In two clinical studies over 60% of men with mild, moderate and severe erectile dysfunction experienced a meaningful improvement in their erections. Eroxon® was shown to work quickly: 63% of clinical trial subjects experienced an erection within 10 minutes in at least 30% of occasions they used the product. 71% of clinical trials subjects were able to have penetrative sex within 15 minutes in at least 30% of occasions they used the product.

### IMPORTANT EDUCATIONAL MESSAGE

Erectile dysfunction may be an early sign of:

- heart and blood vessel problems
- diabetes
- kidney disease
- multiple sclerosis
- liver disease or alcoholism

It is important to have regular check-ups with your doctor because your health may change over time.

## CONTRAINDICATIONS

### DO NOT USE EROXON® IF:

- You have an allergy (hypersensitivity) to any of the product ingredients.

## WARNINGS AND PRECAUTIONS

### STOP USING EROXON® IF:

- You or your partner notice any irritation (e.g. burning, itching or redness). Seek medical attention if the irritation continues.

### DO NOT USE IF:

- You are **under 22**.
- You have any disease or deformity of the penis (such as Peyronie's disease, which causes your penis to be severely bent or curved).
- The skin on your penis is red or sore, or appears damaged or broken.
- You have been advised by your doctor to avoid sexual activity.

### POSSIBLE SIDE EFFECTS

Eroxon® may cause a burning or itching sensation. See Warnings and Precautions section for more information.

### DIRECTIONS FOR USE

Eroxon® is supplied in a single dose tube, that should be stored at room temperature.

#### How to open the tube:

Unscrew cap. Pierce the tube seal with top of cap. Squeeze tube firmly to empty all the contents which will provide the correct dose (approximately a pea-sized amount of gel).

#### How much to apply:

Apply the contents of one tube to the head of your penis.

#### Where to apply:

Apply the gel to the head of your penis.



#### How to apply:

**Massage the gel onto the head of your penis for about 15 seconds.**

If you are uncircumcised, gently pull your foreskin back as far as comfortable before applying the gel.

You can apply the gel, or your partner can apply the gel to your penis as part of foreplay.

#### When to apply:

Relax and be patient if Eroxon® does not help you the first time. You can try again another time. If it has been some time since you were able to get or keep an erection, it may take a couple of attempts before you achieve the desired effect. You may want to practice on your own first. Talk to your doctor if you have tried and are still not able to get and keep an erection.



For more information and to buy online scan QR code or go to Eroxon.us

288 mm

24 mm  
24 mm

## PREGNANCY OR SEXUALLY TRANSMITTED INFECTIONS (STIs)

- Eroxon® is not a contraceptive and contains no spermicidal ingredients. This means that this product will NOT prevent pregnancy in a partner.
- Eroxon® will not prevent STIs. You should use a condom to prevent STIs.
- Eroxon® is compatible with all condoms made from latex and polyisoprene, the materials from which most condoms are made.
- Eroxon® is not compatible with polyurethane condoms.
- Eroxon® has not been tested against contraceptive foams or gels.

## HOW TO STORE EROXON® AND OTHER INFORMATION

Store at room temperature.

Check the expiry date on the tube and carton flap before use. The expiry date refers to the last day of that month.

If the tube is damaged do not use the tube. Throw the tube away and use a new one.

**Ingredients:** Alcohol, aqua, glycerin, propylene glycol, carbomer, potassium hydroxide.

## FREQUENTLY ASKED QUESTIONS

### How does Eroxon® work?

Eroxon® works directly to stimulate blood flow in the penis, which helps you get an erection.

### How quickly does Eroxon® work?

Phase 3 clinical trials showed that Eroxon® helps you get an erection within 10 minutes.

### How long does Eroxon® last?

Eroxon® is shown to help maintain an erection long enough for penetrative sex.

### How often can I use Eroxon®?

You can use Eroxon® as often as required. You should reapply each time you have sex.

### Can I use more than 1 tube at a time?

Eroxon® is proven to be safe and effective when 1 tube is used at a time.

### Is sexual stimulation needed for Eroxon® to work?

Yes, like other products for erectile dysfunction, you need to be sexually stimulated for it to work.

### Is Eroxon® safe for my partner?

Yes.

### Can I have oral sex when using Eroxon®?

Yes.

### Does Eroxon® have a fragrance or flavor?

No, there are no added fragrances or flavor.

### Can my partner apply Eroxon® for me?

Yes, this is a great idea to get you both in the mood.

### Can I use Eroxon® as a sexual lubricant?

No, Eroxon® is not a sexual lubricant and has not been tested in that way.

### Can I use Eroxon® with a lubricant?

Yes, Eroxon® may be used with a sexual lubricant.

### Should I use a condom with Eroxon®?

Condoms are good protection against sexually transmitted infections and in helping to prevent pregnancy. Eroxon® has been shown to be safe for use with most condoms (latex and polyisoprene).

### Can Eroxon® be used as part of foreplay?

Yes, think of using Eroxon® as part of foreplay and try out techniques and positions that may help you achieve an erection. Take your time and enjoy yourselves.

### How do I create the right environment for good sex?

Decide on a time and place when you both know you won't be disturbed and think about how you might achieve some novelty and excitement (e.g. sexy lingerie, scented candles, sex toys). It's a good idea to have some relaxation time before you make love, maybe take a bath or shower together or treat each other to a massage - whatever it is that you like and gives you both pleasure.

### What else can I do to help me get an erection?

Some men require more direct stimulation of the penis than others to achieve an erection, explore other areas of you and your partner's body if it feels good. The important thing is to relax and have fun.

### How can I improve my erectile function?

You can improve your erectile function by making lifestyle changes; these include exercising regularly, reducing stress, giving up smoking and avoiding excessive alcohol use.

### Sometimes anxiety about my sexual performance stops me getting an erection. Will the product work for me?

Yes, it is designed to treat all forms of erectile dysfunction.

### I have premature ejaculation (I ejaculate earlier than I would like to). Should I use Eroxon®?

Only if you have difficulty achieving and maintaining an erection.

### My ED is caused by a significant physical problem. Will Eroxon® work for me?

In a phase 3 clinical study, 31% of men had ED which was assessed as purely psychogenic, and 68% of men were assessed as having organic ED (58% with a psychogenic component and 10% with purely organic ED). The study results indicated Eroxon® was effective in all men including those with purely organic ED but the relatively low number of men in this group may mean that men with purely organic ED may not respond as well as other men.

### I have had a radical prostatectomy. Will Eroxon® work for me?

The effectiveness of Eroxon® has not been studied in radical prostatectomy patients. Ask your doctor for advice about treatments for erectile dysfunction suitable for you.

### What other treatments are available to treat erectile dysfunction?

Alternative treatments include counselling, drug therapy, hormonal therapy, vascular surgery, and implanted prosthetic devices. Contact your doctor for more information regarding these alternative treatments.

Distributed by: Haleon, Warren, NJ 07059  
Pat. Info [www.productpats.com](http://www.productpats.com)

Trademarks are owned by or licensed to the Haleon group of companies.  
©2024 Haleon group of companies or its licensor.

Version: 0.7 Date: 31JAN2024